



(poly-ureaurethane, 16%)

NAIL SOLUTION

INTENDED USE

Nuvail™ is indicated for managing signs and symptoms of nail dystrophy, i.e., nail splitting and nail fragility, for intact or damaged nails. Nuvail coats and adheres to the nail surface preventing direct abrasion and friction on the nail surface while also providing protection against the effects of moisture.

PRODUCT DESCRIPTION

The product is a biocompatible, polymeric solution which forms a uniform film when applied to the nail. The product is dispersed in a non-cytotoxic solution which dries rapidly, adhering to the contours of the nail to form a flexible, waterproof barrier. The film will wear off naturally and must be re-applied as directed to be effective. The film is colorless, transparent and possesses good moisture vapor permeability.

INGREDIENTS

16% Poly (urea-urethane) in organic solvents.

CONTRAINDICATIONS

Do not apply directly to deep, open, or profusely bleeding wounds.

WARNINGS AND PRECAUTIONS

The product is flammable in liquid form; avoid using near open flames and sources of ignition. Use in well-ventilated area. **Keep out of reach of children.**

Store at room temperature away from heat. Do not allow product to come into contact with floors, counter tops, furniture or other finished surfaces - will stain.

May temporarily sting upon application. Persons sensitized to isocyanate should not use this product. Should redness or other signs of irritation appear, discontinue use and consult your healthcare provider.

Use of other products, ointments, creams or lotions before application of this product may prevent the film from forming correctly and reduce effectiveness.

DIRECTIONS FOR USE

- Application is recommended once a day before bedtime
- Remove bottle cap, dip applicator into product and wipe off excess on lip of bottle
- Apply in even strokes to affected nail plate [in entirety], proximal and lateral folds, and the distal tip
- Allow the product to dry completely before applying pads or clothing
- Replace cap with applicator on bottle when not in use
- Clean affected nails with nail polish remover once a week

HOW SUPPLIED

Nuvail Nail Solution is supplied in a: 15 ml non-sterile bottle with applicator 68712-044-01.

STORAGE AND SHELF LIFE

For best results the product should be stored at room temperature (15° to 30°C, 59° to 86°F).

Avoid storage in excessive heat and humidity. Refer to the bottle label for expiration date.

Manufactured for:

Cipher Pharmaceuticals US LLC

Charleston, SC 29492

Toll Free: 1-800-499-4468

www.cipherpharma.com

www.Nuvail.com

S.Code: 68712-044-01

March 2016

This product may be covered by one or more of the US Patent Nos. 7,008,997/8,771,725/9,259,436.

cipher
PHARMACEUTICALS

Rx Only
Nuvail[™]
(poly-ureaurethane, 16%)
NAIL SOLUTION

DETAILED PATIENT INSTRUCTIONS



1. Before starting treatment, remove any loose debris or nail material using nail clippers or a nail file. If you have diabetes or problems with numbness in your toes or fingers, talk to your health care provider before trimming your nails or removing any nail material.



2. Dip the applicator tip into the bottle of NUVAİL and wipe off the excess on the rim of the bottle. One dip can cover all 10 fingernails or toenails.
3. Apply Nuvail nail solution once daily (preferably at bedtime) to all affected nails. Apply a thin layer of the solution evenly over the entire nail (do not over apply).
4. Allow nail solution to dry (approximately 2-3 minutes) before putting on socks or stockings.



5. Clean the affected nail(s) once a week (more frequent if needed), using regular nail-polish remover and regularly file the nails to remove rough, loose edges.
6. Repeat process (steps 2 through 5).

Please Note:

1. Remove and discard the original screw cap. Use the supplied cap with applicator tip to close and store during treatment.
2. To prevent the solution from drying out, bottle should be closed tightly after every use.

cipher
PHARMACEUTICALS