

Unique Power of Nuvail

Functions of Healthy Nails:

- Protect fingertips and surrounding tissue from injury
- Protect nail bed from bacteria and fungi
- A tool for gripping, cutting, scraping

Causes of Damaged, Weak, Brittle Nails:

- Fungal infection
- Aging
- Psoriasis
- Excessive hand washing
- Contact with harsh chemicals
- Environmental triggers
- Injury or trauma to the nail

Benefits of Nuvail:

- Improves nail appearance²
- Goes on clear
- Is easy and convenient
- Takes seconds to apply and minutes to dry

Nuvail™ is indicated for managing signs and symptoms of nail dystrophy, i.e., nail splitting and nail fragility, for intact or damaged nails. Nuvail coats and adheres to the nail surface preventing direct abrasion and friction on the nail while also providing protection against the adverse effects of moisture.

Do not apply directly to deep, open, or profusely bleeding wounds.

Please see Full Important Safety Information on back.



Pay \$0 for Commercially Covered Patients

Nuvail™ is indicated for managing signs and symptoms of nail dystrophy, i.e., nail splitting and nail fragility, for intact or damaged nails. Nuvail coats and adheres to the nail surface preventing direct abrasion and friction on the nail while also providing protection against the adverse effects of moisture.

IMPORTANT SAFETY INFORMATION

Do not apply directly to deep, open, or profusely bleeding wounds. Product is flammable in liquid form; avoid using near open flames and sources of ignition. Use in well-ventilated area. **Keep out of reach of children.** Store at room temperature away from heat. Do not allow product to come into contact with floors, counter tops, furniture or other finished surfaces – will stain. May temporarily sting upon application. Persons sensitized to isocyanate should not use this product. Should redness or other signs of irritation appear, discontinue use and consult your healthcare provider.



For Full Prescribing Information, please visit <http://nuvail.com> or scan the QR code.

REFERENCES: 1. Symphony Health, June 2015. 2. Nasir A, Goldstein B, van Cleef M and Swick L. Clinical Evaluation of safety and efficacy of a new topical treatment for onychomycosis. *J Drugs Dermatol.* 2011; 10(10):1186-1191. 3. Nuvail™ (poly-ureaurethane, 16%) NAIL SOLUTION Prescribing Information. Innocutis, Charleston, SC, 29401 USA; June 2012. 4. Data on File.

cipher
PHARMACEUTICALS

cipherpharma.com

NUV-PEBD-0116

Nail Power

Build strong and healthy nails with Nuvail™



The #1 prescribed brand in dermatology for nail dystrophy¹

Rx Only
Nuvail™
(poly-ureaurethane, 16%)
NAIL SOLUTION

Applying Nuvail


Steps for Proper Application:

1. Dip applicator and wipe off excess on the lip of the bottle.
2. Apply a very thin layer of Nuvail, covering the entire nail.
3. Allow the nail to dry completely. If nails feel sticky, too much product was applied.
4. Replace cap on bottle when not in use.



- Apply Nuvail once a day before bedtime
- Remove Nuvail once a week with fingernail polish remover
- Continue to apply Nuvail until nails are fully regrown and healthy

How Nuvail Works

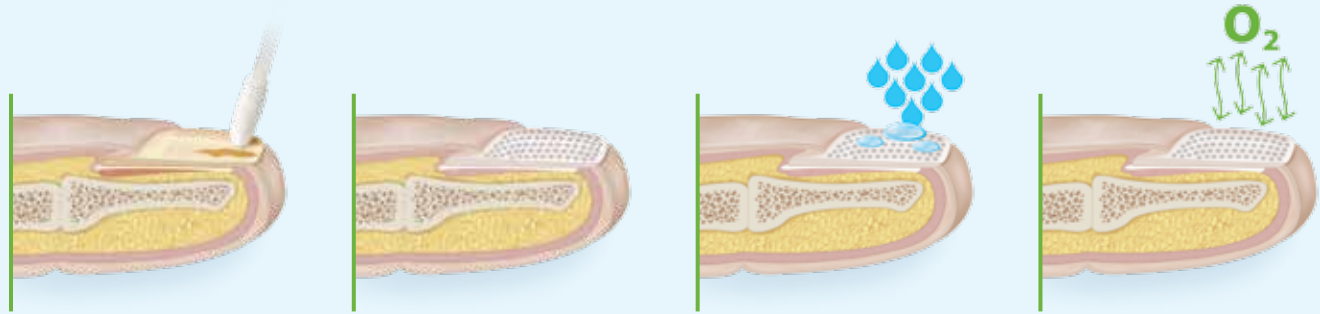

- Penetrates nail surface ridges and fills spaces³
- Promotes a more even, uniform nail plate⁴

- Forms a barrier that is moisture-resistant, yet breathable³
- Reinforces the nail's mechanical support³

What to Expect with Nail Regrowth



Start of treatment

6 months with Nuvail

- It takes 4-6 months on average to completely regrow a fingernail²
- Nuvail can improve the look and appearance of your nails as they regrow²

In clinical studies, Nuvail was shown to promote improved appearance of the nail in as early as

4 Weeks⁴